



## H-HA+L-HA+PDRN+PN Skin Booster

### Ingredients

High molecular weight HA: 67.5mg  
 Low molecular weight HA: 67.5mg  
 PDRN(Polydeoxyribonucleotide): 5mg  
 PN(polynucleotide): 5mg

### Indication

Enhance Moisture  
 Long-lasting Volume  
 Wrinkle Reduction  
 Cell Regeneration  
 Anti-Inflammatory  
 Collagen & Elastin Regeneration  
 Improvement of Skin Texture  
 Skin Elasticity  
 Enhance Wound Healing  
 Scar & Acne Improvement

### Treatment Area

Forehead, Cheek, Jaw, Chin, Hollow temple, Tear troughs, Nasolabial folds, Marionette lines, Neck, Bunny lines, Mental crease, Jowls, Back of hand

### Advantages

Long lasting  
 Evenly restores volume & improves wrinkles  
 Synergistic effect of lifting and collagen regeneration  
 Quick procedure and effect  
 No residues left in the bod  
 Full face treatmen  
 FDA Proven Safety  
 Wound healing

### Treatment

3 Treatments (2 weeks interval)

### Duration

4~8 Months

### Volume

5 ml / vial / box



### 01

#### H-HA+L-HA+PDRN+PN

- Non Crosslinked H-HA+L-HA
- PDRN
- PN

### 02

#### High molecular weight HA & Low molecular weight HA


- Enhanced Moisture
- Long-lasting Volume
- Wrinkle Reduction

### 03

#### PDRN, PN

- Cell Regeneration
- Anti-Inflammatory
- Collagen & Elastin Regeneration
- Improvement of skin texture
- Skin Elasticity
- Enhance Wound Healing
- Scar & Acne Improvement

Action and composition

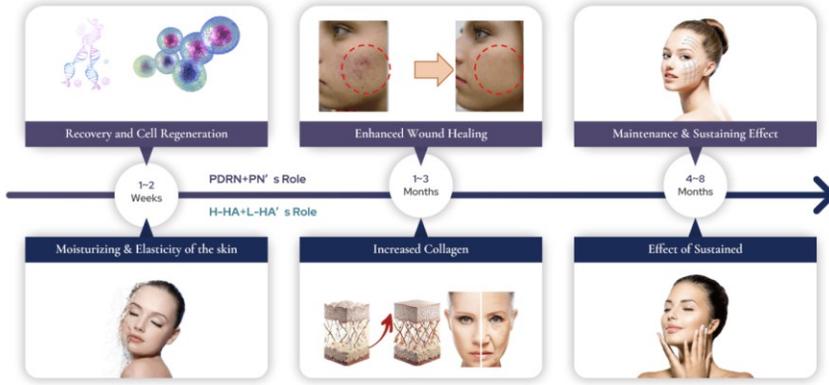


Where do you inject SYNERFILL H-HA+L-HA+PDRN+PN?

- 1 Inject into dermis layer, injection depth is 2mm (34Gx2mm), Inject 0.25cc~0.3cc per point
- 2 Inject 1.0 ml of Lidocaine(2.0%) into 5ml of vial solution. Inject into subcutaneous layer(BAP point). Injection depth is 4~6mm and injection amount is 0.4cc~0.5cc per point



Before and After (2 treatments at 2 weeks intervals)

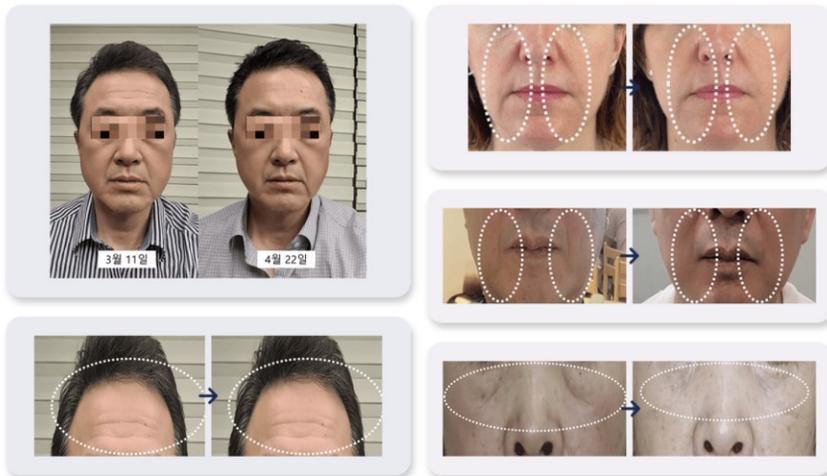

Before and After (2 treatments at 2 weeks intervals)

